

ManaEZ[®] Tendon Strap

MD

INSTRUCTIONS FOR USE

DESCRIPTION OF SYMBOLS

MD Medical Device

PRODUCT DESCRIPTION

The ManaEZ Tendon Strap Universal is an easy to use, quick wrap knee orthosis designed to improve knee cap tracking and reduce the load on the tibial tendon. Decreasing the load reduces patellar discomfort and allows the patient to continue with activities. Wrap around design makes application and removal easy.

INDICATIONS FOR USE

Ideal for Patellar tendinitis, Patella tracking disorders, Osgood-Schlatter disease, and Patellofemoral knee pain.

GENERAL SAFETY INSTRUCTIONS / WARNINGS AND PRECAUTIONS

Strong supervision when using with children and geriatric patients. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

CLEANING AND CARE

Hand wash with mild detergent and let air dry. do not machine wash or dry.

SYMBOL GLOSSARY

For an explanation of symbols, please refer to www.manamed.com/glossary/symbol-glossary.pdf

FITTING INSTRUCTIONS



1. Open the brace by pulling apart the Hook & Loop.



2. Wrap the brace around the front of the leg, just below the patella



3. Attach the Hook & Loop behind the knee, just below the patella. Walk around. Adjust tension as necessary.



MANAMED

ManaMed, LLC
2612 Sirius Dr.
Denton, TX 76208
Ph: 888-508-0712
www.Manamed.com