

ManaEZ[®] Knee Immobilizer Three Panel


INSTRUCTIONS FOR USE



DESCRIPTION OF SYMBOLS



MD Medical Device

SPLINT PANEL PLACEMENT:

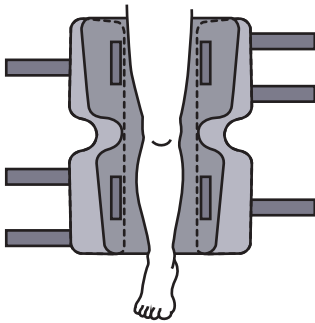



1. Place the immobilizer under or behind the leg. Center the leg on the rear stay.

2. Detach the removable panels and adjust laterally for correct sizing of thigh and calf circumference.

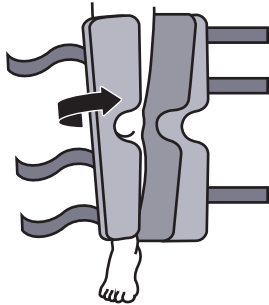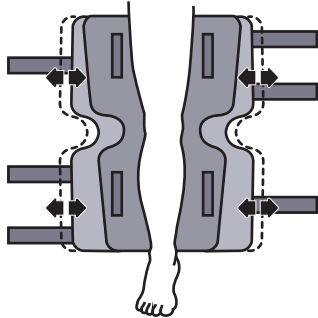

3. For a proper fit the patella (kneecap) will be in the cutout in the center of the brace, when properly positioned wrap the panels around the leg and attach to the center panel.

4. Starting with the upper calf strap. Insert the straps through the D-rings and pull to the desired tension. Secure with the hook tabs provided. Then secure the lower thigh strap. The remaining straps work your way up the leg from the foot to the thigh. Trim straps if necessary.

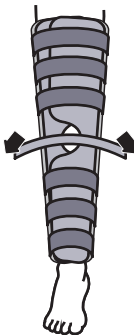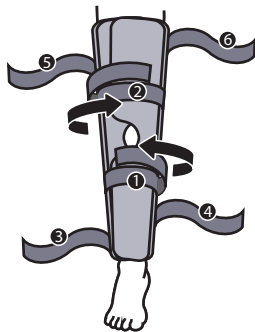

5. If prescribed, secure the patella strap over the center of the knee joint by placing the hook squares on sides of knee and attach strap.

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

Immobilization of the knee joint.

CONTRAINDICATIONS

None.

WARNINGS AND PRECAUTIONS

If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the support over an open wound.

APPLICATION INFORMATION

Note: The Foam Pop Pad is included with ManaEZ Knee Immobilizer and may be used for comfort and to help prevent migration. If the Foam Pop Pad is not applied to your splint, please see Foam Pop Pad Placement instructions below.

FOAM POP PAD PLACEMENT

Center pad over the stays located on the posterior (back) panel of immobilizer. Pad should be over the bend in the stays. Remove adhesive back on pad and apply to inside of knee immobilizer. Foam pad may provide additional comfort and support behind the knee.

CLEANING INSTRUCTIONS

Hand wash liner in warm water using mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. For Knee Immobilizers with removable stays, remove stays prior to washing.

WARRANTY

ManaMed will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

SYMBOL GLOSSARY

For an explanation of symbols, please refer to <https://www.manamed.com/glossary/symbol-glossary.pdf>

NOT MADE WITH NATURAL RUBBER LATEX.

FOR SINGLE PATIENT USE ONLY.

Product Name	Part Number
ManaEZ 3 Panel Knee Immobilizer 16"	EZ3PIMM01-16
ManaEZ 3 Panel Knee Immobilizer 18"	EZ3PIMM01-18
ManaEZ 3 Panel Knee Immobilizer 20"	EZ3PIMM01-20
ManaEZ 3 Panel Knee Immobilizer 24"	EZ3PIMM01-24



MANAMED
ManaMed, LLC
2612 Sirius Dr.
Denton, TX 76208
Ph: 888-508-0712
www.ManaMed.com

©2024 ManaMed LLC. All rights reserved.
ManaMed, the ManaMed logo,
are trademarks of ManaMed LLC
Registered in the U.S.