



Universal Left or Right Knee Brace Instructions for Use

The GOAT is a universal OA brace that allows for a varus or valgus tilt in one brace. Indicated for patients with knee arthritis.

Examples of Use

Unicompartmental OA-medial or lateral Chondral defect procedures

Meniscal repairs

Knee DJD

Product Features

Low profile, single upright hinge for daily activities

Butterfly hand operation allows for easy adjustment

Straps and cushions designed for all day support

Lightweight

Universally sized, but left or right specific.



Caution! Please read carefully prior to product use:

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling.

WARNING:

Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.

CLEANING INSTRUCTIONS:

Fasten closures. Hand wash in cold water with mild soap. Wrap in a towel to remove excess water (do not twist or wring). Air dry away from heat. Do not tumble dry. Do not bleach.

Order information

Part Number for GOAT Left Knee: GOA001-L
Part Number for GOAT Right Knee: GOA001-R

How to order

Email: orders@manamed.net, Fax: 888-508-0712, Call: 888-508-0712

5240 W Charleston Blvd.,
Las Vegas, NV 89146
www.ManaMed.net



MANAMED™